



VELOX® FREEDOM TO BREATHE.

FREEDOM TO LIVE.



NEW

Mobile and efficient nebuliser device for therapy of acute and chronic airway diseases

MADE IN GERMANY

www.pari.com/de-en



VELOX® QUICK & EFFECTIVE TREATMENT

The new VELOX makes life easier for patients with diseases of the lower airways. It is both efficient and fast, but also very practical. Small, light and easy to manage, it's also incredibly quiet, and totally mobile.

The VELOX can be incorporated seamlessly into the daily routine, allowing patients to get on with their lives.



*from age
3 years*



VELOX®

Item number: 055G1001



*0-3
years*

VELOX® Junior

Item number: 055G1101



THE ADVANTAGES FOR YOUR PATIENTS IN SUMMARY:

- small
- light
- quiet
- fast
- mobile

PATIENTS WHO SUFFER FROM ACUTE OR CHRONIC DISEASES OF THE AIRWAYS BENEFIT FROM:



fast and **efficient** aerosol therapy due to high aerosol output with a high percentage of droplets able to reach the lungs (approx. 3 min for 2.5 ml salbutamol)



maximum mobility due to both mains and battery power and a compact size



simple handling due to ergonomic design, small number of components and light weight*



discreet treatment due to extremely quiet **operation**




consistently short treatment times due to VELOX[®]care cleaning aid

* Weight: 110 g without batteries / 140 g with batteries




FAST AND EFFICIENT THERAPY VIA MODERN VIBRATING MEMBRANE TECHNOLOGY

Respirable dose (< 5 µm) in %

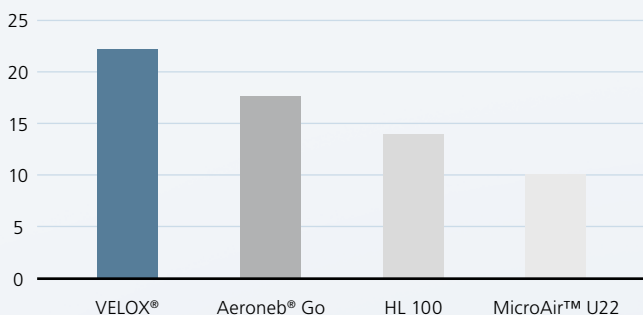

Inhalation time in minutes

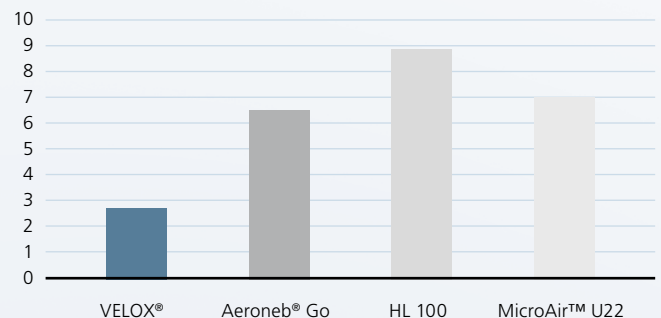

Of the four different nebuliser devices compared, VELOX delivers the highest amount of drug (respirable dose) in the shortest time.

Aerosol characteristics:	
Total Output Rate:	497 mg/min
MMD:	3.8 µm
Mass fractions <5 µm:	74 %
<i>Measurement using laser diffraction (Helos SympaTec) at 23 °C and 50% relative humidity. Nebulised medium: 0.9% NaCl (2.5 ml). Inspiratory flow: 20 l/min.</i>	

Internal measurements, PARI Pharma GmbH, Gräfelfing (Germany). Nebulised medium 2.5 mg / 2.5 ml albuterol adult breathing pattern Vt = 500 ml; 15 bpm; In:Ex = 1:1

VELOX®



VELOX®

Item No. 055G1001



VELOX® Junior

Item No. 055G1101

VELOX® PARTS & ACCESSORIES



VELOX®care

Item No. 055G6100



VELOX® AC power supply (EU)

Item No. 055G7100



VELOX® Bag

Item No. 055E8000



PARI GmbH

Moosstraße 3, 82319 Starnberg, GERMANY

E-Mail: info@pari.de

Tel. +49 (0) 8151 279-0

Fax +49 (0) 8151 279-101

More information at: www.pari.com/de-en